


Electronic Measurement Tool: EYE-RULER™ 2

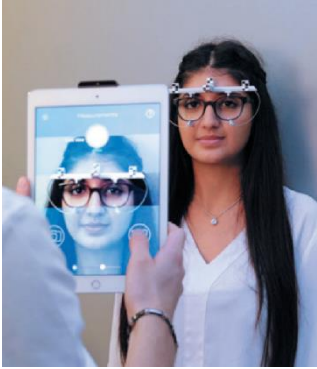

- Measurements: all necessary and personalized parameters are taken with a single solution
- Distance between ECP and patient ≥ 3.3 ft
- The patient does **not need to touch or hold the device** for far-vision measurements.
- The ECP has **no physical contact** with the patient while taking measurements.
- The patient stands at 2.5 ft from the device.



Manual measures



- Distance ECP/patient = **1.1 ft max**
- Device **must be in contact with the patient's face** (nose and forehead).
- Patient needs to hold the device.



- Distance ECP/patient = arm's length
- ECP **risks touching the patient's face** while holding the ruler.
- ECP needs to be close to the patient to read the ruler's markings.



Frame with clip handling: When adjusting the previously-cleaned frame and positioning the clip on the frame, the ECP must wear disposable gloves as well as a protective mask.

Cleaning recommendations: List of items to be cleaned before and after each set of measurements: the frame, the clip, the iPad®*
To clean all electronic devices, we recommend using hydroalcoholic wipes. The clip can be cleaned with hydroalcoholic wipes or a soft, dry cloth with an alcoholic solution.

Warning: When cleaning all electronic devices, do not let any liquid enter electronic parts, please use hydroalcoholic wipes.